
Instructions for Use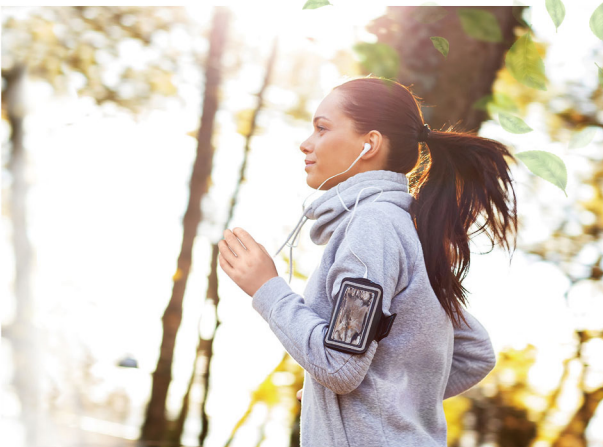

Step Toward Tomorrow

REGAL PROSTHESIS LIMITED

*An ISO 13485:2016 and ISO 9001:2015 certified company
Your reliable partner*

Contents

• EN	Instructions for Use	2
• DE	Bedienungsanleitung	8
• RU	Инструкция по применению	15
• TR	Kullanım Talimatları	22
• UK	Інструкції до використання	28
• ES	Instrucciones de uso	34
• PT	Instruções de utilização	40
• PL	Instrukcja użytkowania	46
• NL	Gebruiksaanwijzing	52
• ZH	使用说明	58
• ZH	使用說明	63
• KO	사용설명서	68
• HI	उपयोग के लिए निर्देश	73
• HE	הוראות שימוש	79
• AR	تعليمات الاستخدام	84
• SV	Bruksanvisning	89
• HR	Upute za upotrebu	95
• DA	Brugsanvisning	101
• NO	Bruksanvisning	107
• FI	Käyttöohjeet	113
• FR	Mode d'emploi	119
• IT	Istruzioni per l'uso	126
• CZ	Návod k použití	133

1. Applications

- REGAL PROSTHESES LIMITED Silicone Cosmetic Prostheses are used in providing aesthetic appearance to amputees so as to build up their confidence and to improve their social quality of life.

2. Indications and Patient Target Groups

<i>Item code</i>	<i>Trade name</i>	<i>Patient Target Groups</i>
101L, 102L, 102LS	Silicone Prosthetic glove	For patients of above wrist amputation
101LL, 102LL, 102LSL	Silicone Prosthetic glove (Long)	For patients of above elbow amputation (long sleeve)
103	Silicone Partial hand glove	For patients of partial hand amputation
100	Silicone Myo/mechanical glove	For patients with either myo-electric or mechanical hands
100L	Silicone Myo/mechanical glove (Long)	For patients with either myo-electric or mechanical hands (long sleeve)
100PL	Silicone Myo glove (Pediatric)	For pediatric patients with myo-electric hand
200 series	Silicone Prosthetic finger	For patients with finger amputation
HDSFHT, HDSF	Silicone Footcover	For patients with partial foot amputation
HDSFHT-ER, HDSF-ER	Silicone Footcover with carbon footplate	For patients with partial foot amputation
Toe series	Silicone Prosthetic toe	For patients with toe amputation
BKSL	Silicone Prosthetic leg sleeve	For patients who need covering the lose of partial calf muscle or covering lower limb prostheses respectively
BKSF	Silicone Prosthetic Leg Sleeve plus footcover	For patients who need covering the lose of partial calf muscle and partial foot amputation, or covering lower limbs prosthesis respectively
BKSF-ER	Silicone Prosthetic Foot-Leg cover with carbon footplate	For patients who need covering the lose of partial calf muscle and partial foot amputation, or covering lower limbs prosthesis respectively
SL	Silicone Prosthetic arm sleeve	For patients who need covering the lose of partial arm muscle
CH	Silicone Child crawling hand	For infant patients with hand amputation
PREM-FING-SP	Premium - Silicone cosmetic prosthesis for fingers	Regal Finger Prosthesis, for patients who require premium silicone cosmetic prosthesis for finger

Item code	Trade name	Patient Target Groups
PREM-LL-SP	Premium - Silicone cosmetic prosthesis for Lower Limbs	Regal Lower Limb Prosthesis, for patients who require premium silicone cosmetic prosthesis for Lower Limbs
PREM-TOE-SP	Premium - Silicone cosmetic prosthesis for Toe	Regal Toe Prosthesis, for patients who require premium silicone cosmetic prosthesis for toe
PREM-UL-SP	Premium - Silicone cosmetic prosthesis for upper limbs	Regal Upper Limb Prosthesis, for patients who require premium silicone cosmetic prosthesis for upper limbs
Semi-Custom Made (SCM)	Silicone cosmetic prosthesis: Semi- Custom Made	Adaptation to residual limb, applies to all above item codes
Fully-Custom Made (FCM)	Silicone cosmetic prosthesis: Full Custom Made	Designed to match with patient's original skin pattern, nail shape, patient's original skin tone and appearance; mimic the sound side's shape and color to create prosthetic devices with a natural outlook
REG-MAXI-C-CHIN	Silicone cosmetic prosthesis: Artificial Chin	For patient who wants to restore physical appearance because of missing a chin
REG-MAXI-C-EAR	Silicone cosmetic prosthesis: Artificial Ear	For patient who wants to restore physical appearance because of missing an outer ear
REG-MAXI-C-NASAL	Silicone cosmetic prosthesis: Artificial Nose	For patient who wants to restore physical appearance because of missing a nose

3. Contraindications

- Our products should not be used in the presence of open wound.

4. Precautions

- Our Silicone Cosmetic Prostheses should be fitted by a certified prosthetist who is familiar with the purpose for which they are prescribed. The prosthetist is responsible for providing wearing instructions and precautions to the amputees.
- Contact your prosthetist if:
 - Unusual swelling, skin irritation occurs, limit blood circulation, stop using the product and consult with the prosthetist immediately;
 - The Silicone Cosmetic Prosthesis has split or any

- damage has occurred;
- Any of the inner structure such as finger wires or wrist connector is broken which may cause injury or damage the prosthesis with continued use;
- The prosthetic socket is broken which may cause injury or damage the Silicone Cosmetic Prosthesis with continued use
- Under any circumstances the silicone prosthesis shows potential safety precaution to the patients should discontinue to use and to consult your certified prosthetists and orthotists immediately.

5. Instructions for Use

- Item code 101L, 102L, 102LS, 101LL, 102LL, 102LSL, CH:
 - Apply water-based lotion onto the prosthetic socket and gently slide the glove onto it. (Consult with your prosthetist for arm socket).
 - For item code 102L and 102LL, epoxy glue can be used at the bottom end of the socket to attach onto the pre-filled foam.
- Item code 100, 100L and 100PL:
 - Apply water-based lotion inside the glove surface before installing the glove on the myo-electric or mechanical hand.
- Item code 103, 200 series, HDSFHT, HDSFHT-ER, HDSF, HDSF-ER, TOE series, BKSL, BKSF and SL:
 - Apply water-based lotion onto the residual limb before donning the prosthesis.
- Item code REG-MAXI-C-CHIN, REG-MAXI-C-EAR, REG-MAXI-C-NASAL:
 - Apply medical adhesive on BOTH the residual limb and the prosthesis, press firmly to the residual limb so that the prosthesis will hold onto the position.

6. Care and Cleaning

- Keep the silicone prosthesis away from fire, solvents and oil-based liquid; DO NOT soak prosthesis with any type of

solvent;

- Avoid using sharp objects such as knives or scissors as these could cut and tear the prosthesis easily.
- Do NOT expose the prosthesis to sunlight for extended periods, this could cause the color to fade.
- If a small split happens at the edge of the prosthesis, please return the prosthesis to your prosthetists for removing the split. Otherwise, the small split will continue to grow and break the whole prosthesis.
- DO NOT use excessive force to pull residual fingers from the silicone prosthesis, this may tear the prosthesis. The prosthesis may create a vacuum between the residual limb and the prosthesis inner surface, in this case, simply pinch the prosthesis and make an air passage to eliminate vacuum before removing the prosthesis.
- Remove stains immediately with mild soap and water. Stains may become permanent if not cleaned immediately, our warranty does not cover such incidents;
- For hygiene purposes, we recommend to cleaning your prosthesis daily with mild soap and water;
- Make use of small items such as bandages, rings, bracelets, sports wrist band or watches to camouflage the prosthesis edge;
- When using a silicone prosthesis with zipper, please ensure the zipper is FULLY opened before donning and doffing. If the zipper is not fully opened while donning or doffing the zipper can be damaged easily; Clean the zipper everyday with water and soft toothbrush, gently brush the zip teeth and remove the dirt;
- For item codes that integrated finger wires, hinges, or wrist connectors: DO NOT apply force such as using the prosthesis to support or to lift the body or carry any object. Do keep the silicone prosthesis away from fire, solvents and oil-based liquid; DO NOT soak prosthesis with any type of solvent; doing so will damage the finger wires or hinges and injury may result from broken finger wires or hinges caused by misuse. DO NOT use our finger prosthesis to

type or carry object. Our silicone prostheses are designed for cosmetic purposes only;

- For all item codes, DO NOT take the inner filling out of the prosthesis as this will void the warranty.
- For upgrade options (Dual Coloring, Custom Coloring, X-series, Acrylic Nails, Smooth Coating, Custom Filling, Fingers Construction and Glue To Socket), please refer to our catalogue and our website (www.regalprosthesis.com).
- For more care and cleaning details, please refer to our catalogue or website (www.regalprosthesis.com), and our YouTube channel.

7. Normalized Symbols

	Consult instruction for use
	Complies with the European Medical Device Regulation (MDR 2017/745)
	Complies with the United Kingdom Medical Device Regulation (UK MDR 2002)
	Manufacturer
	Authorized representative in the European Community
	United Kingdom Responsible Person
	Item code
	Serial number
	Date of manufacture
	Unique device Identifier
	Medical Device
	Caution

8. Storage

- Avoid high temperature, store below 35°C/95°F;
- Avoid moisture. Store below 65%RH;

- Avoid direct sunlight. Store in original package when not used.

9. Warranty and Return Policy

- The product is warranted against defective materials and workmanship for a period of 12 months from the date of installation, conditions applied. For more details, please refer to our catalogue or website (www.regalprosthesis.com).
- Warranty will be VOID:
 - if the product comes into contact with aggressive cleaning agents, salt water, chlorinated water or the like;
 - not following the instructions mentioned above;
 - modifications of the products not expressly approved by the manufacturer

10. Disposal

- Dispose of the device in accordance with the regulation applicable at the place of operation. If you have any queries, please refer to the local authorities responsible for waste disposal.

11. Liability

- The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer. Please keep this manual in safe place for future reference.
- If any serious incident occur in relation to the device, patient or partitioner should report to the manufacturer and the competent authority of the Member State in which the user and/ or patient is established.

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Regal Prosthesis Limited